

CARE & USE GUIDE

SureStep[™] Mono & Mono Low Steps

Thank you for your purchase of the SureStep[™] Mono or Mono Low step. The following are our standard care and use instructions for the SureStep[™] Mono and Mono Low steps. Please feel free to contact the installer of this productor Hawk Labs with any questions you may have.

- Do not use the tub or apply weight to SureStep[™] Mono or Mono Low step for two (2) days after installation to allow for the adhesive to completely dry.
- Prior to first use, please inspect the bead of caulk around the perimeter of the SureStep[™] Mono or Mono Low step to ensure that a continuous caulk bead has been applied and that there are no cracks or areas where caulk is absent. Conduct a similar inspection of the perimeter adhesive every six (6) months thereafter.
- Prior to first use, please clean and rinse the bathtub to rid the area of any dust and/or particulates that may remain after installation.
- Do not fill the tub with water over the opening of the SureStep[™] Mono or Mono Low step.
- It is recommended that the unit be cleaned regularly with a gentle household cleaning agent. Do not use harsh cleansers or abrasive tools to clean the SureStep[™] Mono or Mono Low step.
- It is recommended to completely close the shower curtain prior to turning on water to prevent water from splashing outside the bathtub.
- As with any bathtub, children should not be left unattended in the bathtub.